

ENZY-CLEAN

Enzyme-Cleaner Concentrate

Warn advices:

Skin Contact:

Product is proteolytic (= keratolytic). Contact with skin can cause irritation and should therefore be avoided. Please wear gloves while immersing the instrument
In case of splashes onto skin or into eyes please rinse carefully with plenty of water.

Application concentration:

The recommended application dilution should be followed exactly, to achieve the full cleaning activity. Underdosage should be avoided.

Further treatment:

Ensure that no water from the last rinse after disinfection remains in the instrument, since microbes, in particular Pseudomonas species, can proliferate in this water. These microbes can cause infection to susceptible patients. It is advisable to disinfect the endoscope prior to use, if it has not been used for some time (more than 2 days) .

Shelf life:

The concentrate is stable for 2 years. Please do not use after the expiry date.
The application solution must be discarded within 8 hours after preparation.

Producer:

CosmArt Laboratorium GmbH
Neuwiesenfeld 21
D- 64625 Bensheim
www.cosmart.de